


GentiLance™

Contact Activated Safety Lancet

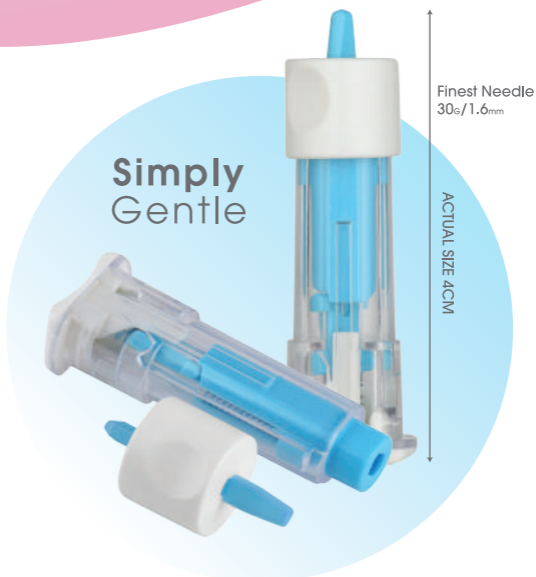

Blood Glucose Monitoring Can Be So Much Easier

Safe

- Automatic sharps retraction system prevents needle from being exposed after use
- Sterilized by gamma irradiation

Gentle


- Unique technology reduces triggering force, sound and vibration
- Obtain the same amount of blood with significantly less pain

Contact Activation


- Consistent penetration depth

Easy


- No pre-loading or pre-arming required
- No lancing device required
- Twist and push

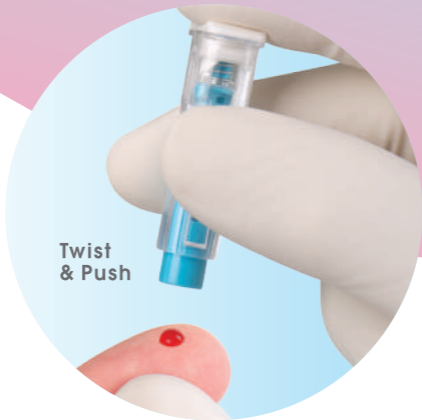


Twist
& Push

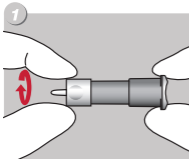


1. Rotate the twisting cap less than half a round

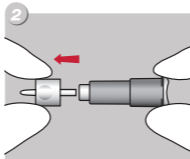


2. Pull out the twisting cap

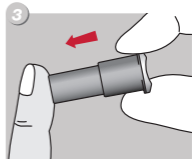



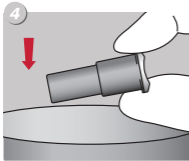

3. Place the device on the puncture site and push to activate

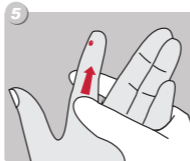

4. Discard lancet into a sharps container



5. Press lightly on the finger towards the puncture site to obtain an adequate blood sample

	Part#	Needle Size	Depth	Blood	Home Use	Hospitals/Clinics
	GL-30-16T	30G	1.6mm	●●●●	✓	
	GL-23-18T	23G	1.8mm	●●●●	✓	✓
	GL-21-22T	21G	2.2mm	●●●●		✓
	GL-21-24T	21G	2.4mm	●●●●		✓

GentiLance™
Contact Activated Safety Lancet

aéras medical

Aeras Medical Pte Ltd
3 Ang Mo Kio Street 62
LINK@AMK, #06-20 Singapore 569139
Tel: +65 6752 0290 | Fax: +65 6752 6109
www.aerasmedical.com